

Clinical Training Preparation



magnes

2026

www.magnes.ch

NUSHU Research App

Download NUSHU Research App

An iPhone or iPad with iOS 16.0 or later is required to install the NUSHU Research app.



Open the app, it will ask permission to find Bluetooth devices. Click on **“Allow”**.



Create an Account

- ❖ Click on "Create account" to create a new user.
- ❖ Fill in all fields.
- ❖ Click "Create".

Fill in all the mandatory* fields.

You will automatically login, the app will ask to allow local networks. Click on **“Allow”**.

Make sure you bluetooth is enabled on your phone, and also in the app settings, as well as local data.

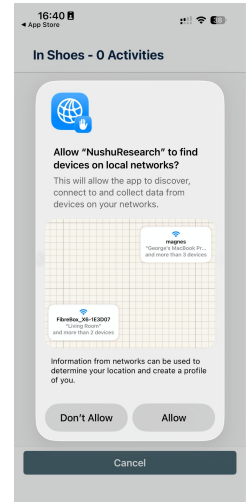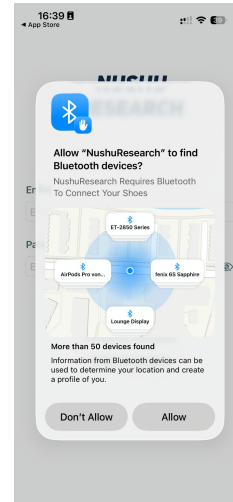


Charge the shoes

Assemble the two pieces of the charging unit like a jigsaw puzzle (2A).

Connect the two charging cables to the power adapter (2B).


Place the shoes on the charging station. Each heel should completely cover a round disc on the charging station (2C).

Fully charging the shoes can take up to **6 hours**.

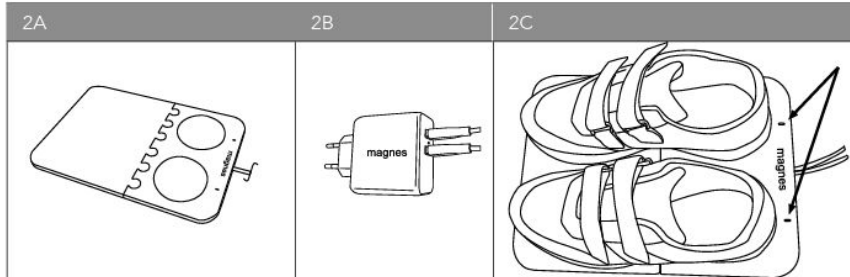

LED Light	Status
SLOW flashing	Charging
QUICK flashing	NOT Charging
OFF	Fully Charged

Create Dashboard account

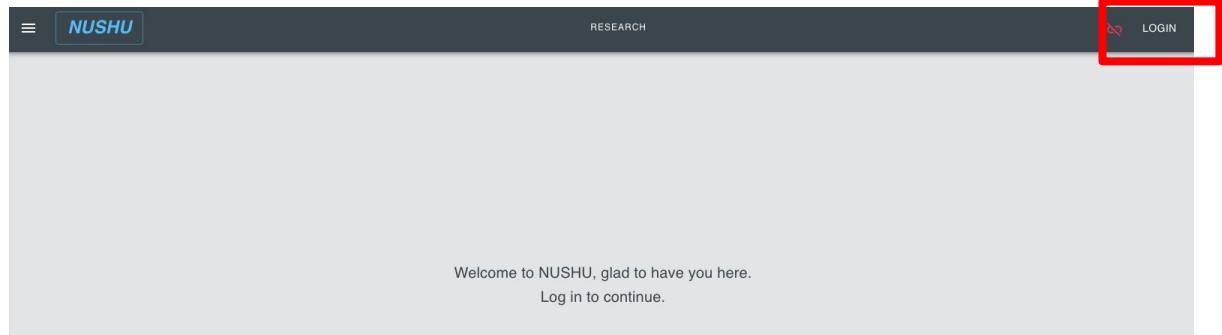

When logging in for the first time, you need to register as a doctor-user.



Every Web Application user is a doctor-user. Users are uniquely identified by their email, so you cannot have a patient-user (mobile app user) and a doctor-user (dashboard) for the same email address. (See next slide, on how to use the same email address)



Make sure your browser has pop-ups enabled to see the login pop-up. Newer versions of Safari, for example, block pop-ups by default, and the user has to give their permission for them to appear.

A login and registration form with a grey border. It contains two input fields: 'Email or Username' and 'Password'. Below these is a 'LOG IN' button. At the bottom, there are two buttons: 'FORGOT PASSWORD' and 'REGISTER'. The 'REGISTER' button is highlighted with a red rectangle.

Alternate email address

A different address that points to the same inbox


It is also possible to create multiple accounts based on one main email address. Assuming the main email address is `nushu@yourcompany.com`, register additional accounts as `nushu::name1@your-company.com`. This way, multiple accounts can be managed through one main email address.

All emails of that account will be sent to `nushu@yourcompany.com`. To log into that account, ensure to use the full email `nushu::name1@yourcompany.com`.
(This can also be used to create patient accounts)

After filling in your email address (this has to be an email address with the clinics domain, [gmail](#) is not possible to use) the right **affiliation** will be selectable.

Click on “**Register**”

You can now **Log In**

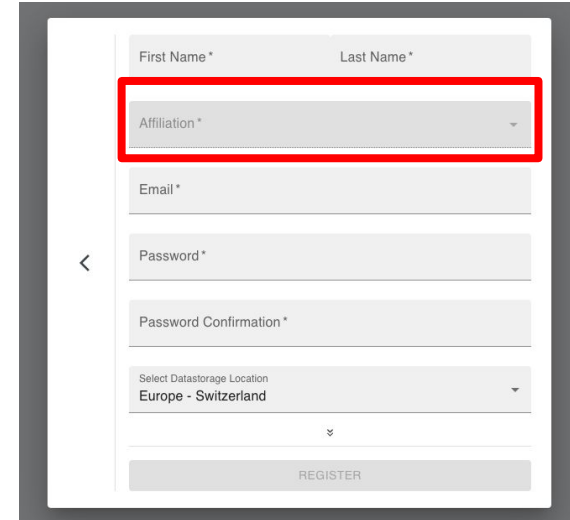

The image shows a registration form with the following fields:

- First Name *
- Last Name *
- Affiliation *** (highlighted with a red box)
- Email *
- Password *
- Password Confirmation *
- Select Datastorage Location: Europe - Switzerland
- REGISTER button